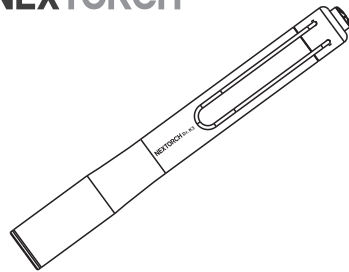
# OPERATOR'S MANUAL 使用说明书 FLASHLIGHT

## NEXTORCH

### DR.K3

Learn more on [www.nextorch.com](http://www.nextorch.com)

#### English

Congratulations on purchasing our NEXTORCH® flashlight. You have chosen a high quality product designed from the latest technology. For the best use of this product, please read the instruction manual carefully.

#### ABOUT NEXTORCH

NEXTORCH Company was established in 2005, a company specializing in the research and development, manufacture and marketing of high end technologically advanced mobile lighting implements. The product range covers medium and high grade flashlights, head lamps, bicycle lights.

NEXTORCH has contributed to the development of the flashlight industry by serving as one of the 14 founding members of PLATO (Portable Lights American Trade Organization) and the voting member of the China Brand Name Flashlight Association, and contributing to the establishment and improvement of ANSI flashlight standards.

When you need a flashlight, a NEXTORCH flashlight will light things up! After ten years of inventions and improvement, requiring reliability of the product quality. NEXTORCH has become one of the leaders of the high end Chinese flashlight market, the leading brand in the European hunting products flashlight market, and among the best of the international name brand flashlight makers, selling to over one hundred countries and regions around the world!

#### ABOUT DR. K3

Dr. K3 is NEXTORCH's first medical flashlight specially targeted for the medical market. Crafted for use in health or medical cases, this flashlight gives high colour rendition and suitable colour temperature which close to the sunshine; The photobiological safety of lamps is RG0(risk group: exempt), without any harm to the eyes; The beam Provides full coverage without dark spots, avoiding mistakes in diagnosis. This 12cm flashlight is designed to fit the pocket of a lab coat, and a fashionable look.

#### SPECIFICATIONS

Bulb	LED
Ra	>90
CCT:	5000 K
Photobiological safety	RG0
Illuminance	950 LUX, 20 cm
Water proof	IPX-7, 1 m
runtime	15 h
battery	1×AAA

#### FEATURES

1. Colour rendition Ra>90, colour temperature approx. 5000 Kelvin, close to observe in the sunshine

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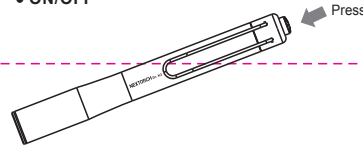
2. Allow to clean or soak with alcohol
3. Photobiological safety of lamps RG0(risk group: exempt), without any harm to the eyes
4. The beam Provides full coverage without dark spots, avoiding mistakes in diagnosis

#### QUICK START GUIDE

##### • Battery Installation

##### • ON/OFF

#### NOTICE

1. Please do not touch with reflector with rough cloths or wipes
2. Remove the batteries during long periods of disuse, to avoid leakage from the battery
3. Please do not remove the light bulb, to avoid harming the reflector cup or the light bulb

#### MAINTENANCE

To keep your flashlight in perfect working condition, follow the instructions below.

- (1) Clean your flashlight with a soft clean cloth.
- (2) Wash the flashlight with fresh water after using in the seawater or causticity liquid, and dry it with warm air.
- (3) When the tail cap does not rotate smoothly, apply silicone grease on the "O" ring (You can buy the silicon grease in outdoor or hardware shops.)
- (4) Please remove the battery and keep them dry and cool when they are not in use for an extended period of time period of time.

#### WARRANTY

1. NEXTORCH warrants our products to be free from any defects in workmanship and/or materials for a period of 15-days from the date of purchase. We will replace it. NEXTORCH reserves the right to replace an obsolete product with a current production, like model.
2. NEXTORCH warrants our products to be free of defect for 5-year of use. We will repair it.
3. The warranty excludes other accessories, but rechargeable batteries are warranted for 1year from the date of purchase.
4. In the event that any issue with a NEXTORCH product is not covered under this warranty, NEXTORCH can arrange to have the product repaired for a reasonable fee.
5. You could access to NEXTORCH website ([www.nextorch.com](http://www.nextorch.com)) to gain warranty service information by scanning the following QR code. You could also: Email us at [service@nextorch.com](mailto:service@nextorch.com) Call us: 0086-400-8300-799 OR contact your local dealer/distributor.

#### ON-LINE SERVICE

Please scan the following QR code, notify NEXTORCH, and we will assign an expert to assist you.

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